



## GAMMA: METS/GBM

This protocol is used for patients that are receiving gamma knife treatment for brain mets or gbm.

- **DO NOT ANGLE**
- Contrast: double dose multihance.
- Run DWI **before** patient is framed.

1. LOCALIZER
2. DWI
3. LOCALIZER
4. AXIAL T2 FATSAT
5. AXIAL FLAIR
6. AXIAL T1 PRE 5mm
7. GADO AXIAL T1 2mm
8. GADO CORONAL T1 2mm MPR